




311 Turner Road
Williamston, MI 48895
Tel: 800-219-9561



Model: 2010, RIFL Blade
DISPOSABLE SINGLE USE
Qty: 1 Pouch = 1 piece

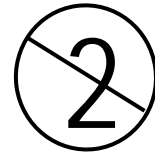

Single Use Only

Prescription Statement

USA Federal Law restricts this device to sale by, or on the order of, a physician or other appropriately licensed medical professional.

Intended Use

The RIFL Blade is a specialized tongue depressor that may be used to depress the tongue for visualization of the oropharynx or as a guide for a flexible laryngoscope or bronchoscope.

Pretreatment

The patient should be prepared for the procedure in the normal regimen.

Directions for Use

- **For visualization of the oropharynx:**
 1. Depending upon the preference of the user either end of the device may be used to depress the tongue.
 2. Open the patient's mouth and insert the RIFL Blade in the midline.
 3. The tongue is then depressed allowing for visualization of the oropharynx.

- **For use in combination with an endoscope:**
 1. The device is placed in the oropharynx in the same way as for visualization of the oropharynx above.
 2. The RIFL Blade is held in the left hand and the tongue is gently depressed opening a channel for the scope to follow.
 3. The endoscope is placed into the oropharynx and follows the integrated channel posterior.
 4. The epiglottis and vocal cords can be visualized as the endoscope is advanced.
 5. The endoscope may then be passed through the vocal cords and a lubricated endotracheal tube may be passed over the scope facilitating endotracheal intubation.
 6. The RIFL Blade and endoscope can now be removed from the patient's mouth.
 7. The endotracheal tube is secured and placement is confirmed.
 8. The patient can now be ventilated by bag valve mask, ventilator or anesthesia circuit.